3-in-1

THERAPEARL®

BREAST THERAPY

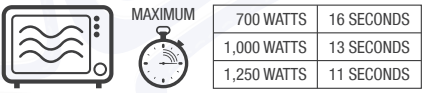
Hot or Cold Breast Therapy

Terapia Mammaia con frío o calor

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BB *Breastfeeding is the most natural gift you can give your baby. However, as natural as it is, breastfeeding sometimes presents challenges. Lansinoh® brings you advanced relief with TheraPearl 3-in-1 Breast Therapy, reusable and non-toxic packs that are latex, lead and BPA free and can be used hot or cold for a range of conditions.*

Intended Use: Use Lansinoh® Therapearl hot to help relieve plugged ducts and mastitis, and when engorged, just before feeding to help let-down and milk flow. It can also be used before or during pumping sessions to reduce the total time spent expressing by encouraging a faster let-down. Use Lansinoh® Therapearl cold to help provide relief from pain and swelling due to engorgement or mastitis. To hold them in place, packs can be placed in a bra.

For heat therapy: Place one pack inside purple cloth cover and microwave lying flat for time indicated in the table (700 Watts – 16 seconds/ 1,000 Watts – 13 seconds/ 1,250 Watts – 11 seconds). Each pack should be heated separately. After testing pack temperature, you may add more time in 5 second increments as needed until desired temperature is reached.

For cool therapy: Freeze for at least 2 hours then place into covers before use.

For use with a pump: Heat packs according to heat therapy instructions and wrap around the outside of breast pump funnels, securing with the fastener, while pumping.

- For external use only - do not use for purposes other than those stated above
- Lansinoh® Therapearl should only be applied for maximum 20 minutes
- Clean with a mild soap, detergent or alcohol
- Do not place any heavy objects on top of the packs
- Do not use on infants or allow infants to chew on the packs

CAUTION: Do not heat Lansinoh® Therapearl packs beyond the time stated. Heating beyond suggested time may damage or destroy Therapearl pack. Always test pack temperature prior to application. Overheating or prolonged application may cause injury. Do not apply to sensitive skin or open wounds. Use only under adult supervision. Do not puncture pack and/or ingest contents. Do not fall asleep using pack. For extreme, prolonged or chronic pain, consult a physician.

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